





# SDMA



## Sample Collection and Submission

Assay	Species	Platform	Sample Type	Sample Volume
SDMA	Cat	Beckman	Serum, Plasma	80 µL
SDMA	Dog	Beckman	Serum, Plasma	80 µL
SDMA	Rat	Beckman	Serum	80 µL

### Serum, Plasma Samples

	Use serum separator or lithium heparin tubes, pipette serum or plasma into sterile transportation tubes and place those into sealable plastic bags
	Label bag(s) with sample type, submitter name and study number if applicable.
	Samples will remain stable if stored at 4°C for 7 days
	If banking and freezing samples, store at -20°C to -80°C

### Prepare for Shipping

	Include a copy of the submission form with samples.
	Samples stored at 4°C can be shipped on ice/ice pack Samples banked and frozen can be shipped on dry ice to avoid freeze/thaw

### Ship Samples to:

IDEXX BioAnalytics  
3 Centennial Drive  
North Grafton, MA 01526

IDEXX BioAnalytics  
2825 KOVR Drive  
West Sacramento, CA 95605

### Customer Support Services

800-669-0825 • [idxxbioanalytics@idexx.com](mailto:idxxbioanalytics@idexx.com)  
[idxxbioanalytics.com](http://idxxbioanalytics.com)