





Sample Collection and Submission



Assay	Species	Platform	Sample Type	Sample Volume*
SDMA	Rat	LCMS	Serum, Plasma	50 µL
SDMA	Mouse	LCMS	Serum, Plasma	50 µL
SDMA	NHP	LCMS	Serum, Plasma	100 µL
SDMA	Dog	LCMS	Serum, Plasma	100 µL
SDMA	Cat	Beckman	Serum, Plasma	100 µL
SDMA	Dog	Beckman	Serum, Plasma	100 µL
SDMA	Rat	Beckman	Serum, Plasma	100 µL

* Minimum volume required

Serum, Plasma Samples

	Use serum separator or lithium heparin tubes, pipette serum or plasma into sterile transportation tubes and place those into sealable plastic bags
	Label bag(s) with sample type, submitter name and study number if applicable.
	Samples will remain stable if stored at 4°C for 7 days and at least 5 years at -20°C to -80°C
	If banking and freezing samples, store at -20°C to -80°C

Prepare for Shipping

	Include a copy of the submission form with samples.
	Samples stored at 4°C can be shipped on ice/ice pack Samples banked and frozen can be shipped on dry ice to avoid freeze/thaw

Ship Samples to: IDEXX BioAnalytics
3 Centennial Drive
North Grafton, MA 01526

IDEXX BioAnalytics
2825 KOVR Drive
West Sacramento, CA 95605

LCMS Only—Ship to:
IDEXX BioAnalytics
Lab 57
One IDEXX Drive
Westbrook, ME 04092

Customer Support Services
800-669-0825 • idxxbioanalytics@idexx.com
idxxbioanalytics.com